




Pressure-Flow Curve

DeltaWave Nasal Pillow System - Exhaust Flow

	4 cm H ₂ O	8 cm H ₂ O	12 cm H ₂ O	16 cm H ₂ O	20 cm H ₂ O
Small (Avg)	18.4	28.0	36.2	43.1	49.6
Med (Avg)	22.0	33.8	43.5	51.8	59.6
Large (Avg)	26.2	39.4	50.4	59.7	68.6

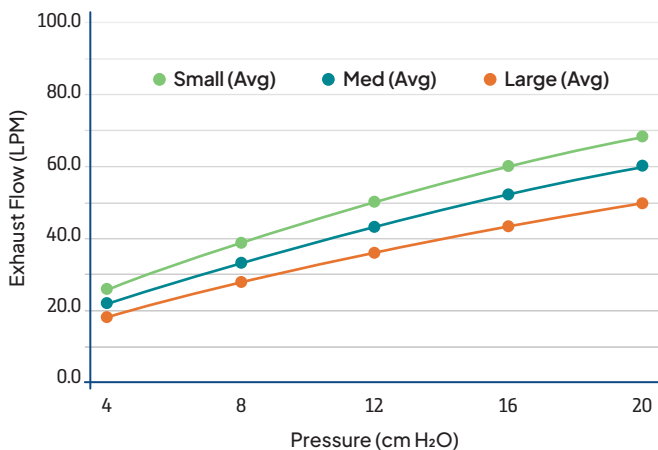

Therapy pressure range: 4–20 cm H₂O

CO₂ Rebreathing

The interface contains passive venting to protect against re-breathing.

		CO ₂ Rebreathing		
		Baseline ET CO ₂ %	Mask ET CO ₂ %	Relative Increase %
Small (Avg)	4cm H ₂ O	5.1	5.2	2
	5cm H ₂ O		5.2	1
	10cm H ₂ O		5.1	0
	Open End Occluded			
Medium (Avg)	4cm H ₂ O	5.2	5.2	0
	5cm H ₂ O		5.2	1
	10cm H ₂ O		5.2	0
	Open End Occluded			
Large (Avg)	4cm H ₂ O	5.3	5.3	1
	5cm H ₂ O		5.3	0
	10cm H ₂ O		5.3	0
	Open End Occluded			

Resistance Drop*

Pressure drop at given flow rates (cm H ₂ O)		
Mask Size	Flow Rate: 50L / min	Flow Rate: 100L / min
Small (Avg)	0.4	1.5
Medium (Avg)	0.2	1.0
Large (Avg)	0.2	0.8

* In pressure measured (nominal) at 50 L/min and 100 L/min



Ordering & Resupply Made Simple

DeltaWave simplifies everything from first fit to final billing.

HCPCS and Ordering Reference – DeltaWave Nasal Pillow System

HCPCS	Replacement Schedule	Description	DeltaWave Part #
A7034 + A7035		DeltaWave Nasal Pillow Start Kit (all 3 pillow sizes)¹	8000
A7034 + A7035		Nasal Pillow System – Single Pillow Configuration²	
	1 per 3 or 6 months	Small Nasal Pillows System	8011
	1 per 3 or 6 months	Medium Nasal Pillows System	8012
	1 per 3 or 6 months	Large Nasal Pillows System	8013
A7034		Nasal Interface – Single Pillow Configuration³	
	1 per 3 months	Small Nasal Interface (Mask without headgear)	8008
	1 per 3 months	Medium Nasal Interface (Mask without headgear)	8009
	1 per 3 months	Large Nasal Interface (Mask without headgear)	8010
A7033		Nasal Pillows – Replacement	
	2 per month	Small	8001
	2 per month	Medium	8002
	2 per month	Large	8003
A7035	1 per 6 month	Headgear	8005
		Accessories	
		Hose Assembly	8004
		Angled Adapter	8006
		Swivel Adapter	8007




Contact Us

Want to learn more or try DeltaWave for your patients?

Call REMSleep at **912.590.2001** to connect with your local representative.

Visit: www.remsleep.com

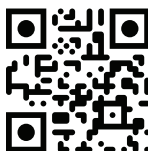

¹ Starter Kit components: Headgear x 1; Nasal Pillows x 3 (1 Pillow in each size: Small (SM), Medium (MD), Large (LG)); Hose Assembly x 1; Swivel Adapter x 1; Angled Adapter x 1; User Guide x 1

² Nasal Pillow System components: Headgear x 1; Nasal Pillow size x 1; Hose Assembly x 1; Swivel Adapter x 1; Angled Adapter x 1; User Guide x 1

³ Nasal Interface components: Nasal Pillow size x 1; Hose Assembly x 1; Swivel Adapter x 1; Angled Adapter x 1; User Guide x 1